# TENS and EMS Muscle Stimulator

## AS8012B

English Version .....	1-34
German Version .....	35-70
French Version .....	71-104
Italian Version .....	105-136
Spanish Version .....	137-167

# Pain Relief, Anywhere, Anytime

Please read the product introduction carefully  
before using the product

MedNet EC-REP GmbH  
Borkstrasse 10,48163 Muenster · Germany

Manufactured by :Shenzhen Astec Technology Co.,Ltd

Address: 8E XinBaoYi Industrial Bld,Houting Village, BeiTing Rd,Shajing,  
Baoan,ShenZhen. Guangdong 518104 China.

Version: ASDV1.0

# Contents

<b>1. Introduction .....</b>	<b>4</b>
Foreword .....	4
Symbols.....	5
Accessories .....	6
Cautions .....	6
<b>2. Operation of the device and Feature .....</b>	<b>11</b>
Operating Procedures.....	12
Features .....	15
Battery Charger Information.....	16
<b>3. Product Specification .....</b>	<b>17</b>
<b>4. Modes Description .....</b>	<b>18</b>
<b>5. Placement of Electrodes.....</b>	<b>23</b>
<b>6. Troubleshooting .....</b>	<b>25</b>
<b>7. Maintenance and Storage .....</b>	<b>27</b>
<b>8. Electromagnetic Compatibility .....</b>	<b>28</b>
<b>9. Warranty .....</b>	<b>34</b>

## Customer Service:

 [support@belifu.com](mailto:support@belifu.com)

- Please download the PDF manual on the below link:  
<https://belifu.com/downloads/>
- Please read the instructions manual carefully before using the device. If you have any questions, please refer to Belifu official, or consult your doctor.
- Please keep this manual for future reference.

## 1. Introduction

### Foreword

AS8012B is a TENS EMS machine with two output channels and is highly effective in relieving pain. Different program suits different parts of the body and pain. TENS stands for Transcutaneous Electrical Nerve Stimulation is recognized as a safe and highly effective method for pain relief. TENS is now regularly recommended by doctors, physiotherapists and pharmacists throughout the world. EMS modes can effectively help build muscles and improve the muscle performance.

## Symbols

Symbol	Meaning
	Attention, consult accompanying documents
	BF Isolated Applied Part
	Class II equipment
	Disposal in accordance with Directive 2002/96/EC
	MANUFACTURER
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	Pls read the user manual before use
	CE Mark
IP22	<p>IP Classification: The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm <math>\Phi</math>, 80 mm length, shall have adequate clearance from hazardous parts, and protected against solid foreign objects of 12.5 mm <math>\Phi</math> and greater.</p> <p>The second number 2: Protected against vertically</p>

	falling water drops when enclosure is tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.
--	--

## **Accessories**

- Massage Device x 1pcs
- Large electrode pads x 2pcs
- Small electrode pads x 2pcs
- Middle electrode pads x 6pcs
- Lead wire with 2 connectors x 2pcs
- Lead wire with 4 connectors x 1pcs
- USB cable x 1pcs
- Storage bag x 1pcs
- Instruction manual x 1pcs

## **Cautions**

The intensity of the stimulation is felt differently by each individual and also will vary with physical conditions. Therefore, adjust the strength level to the position you feel most comfortable. Do not assume that by increasing the intensity of the stimulation there will be a stronger effect. However, it is possible that excessive stimulation to the skin will result in irritation or red eruptions.

This product needs special precautions regarding EMC and needs

to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.

1. Do not apply the massage pads near the heart, on the head, above the neck, in the pubic region, over Scarred areas, on the throat or over the mouth.
2. Avoid touching the pads when the unit is on.
3. Pregnant women must avoid using this massager.
4. Do not use this Stimulator if you a pacemaker or other life support equipment such as an artificial lung or respirator.
5. Should not be used while driving, operating machinery, or during any activity in which involuntary muscle contractions may put the user at undue risk or injury.
6. Caution should be used for patients with suspected or diagnosed epilepsy.
7. Caution should be used in presence of the following.
  - When there is a tendency to haemorrhage following acute trauma or fracture;
  - Following recent surgical procedures when muscle contraction may disrupt the healing process
  - Over the menstruating or pregnant uterus
  - Over areas of the skin which lack normal sensation
8. Do not use if you have following medical conditions:
  - Patients with cardiac problems

- Patients with carotid sinus. Tens may cause bradycardia
- Directly over any wound
- Over the uterus of a pregnant woman
- Unless for labour pain
- Over the eye
- Patients with pacemakers
- Over poorly enervated areas
- Over temple
- On or near the trigeminal nerve if you have a history of herpes zoster induced trigeminal neuralgia(Postherpetic neuralgia)

9. Should ask the doctor about this device before using it if the patient:

- Suffer from acute diseases
- Have a tumour
- Have an infectious disease
- Suffer from fever
- Have blood pressure troubles
- Have skin diseases
- Recently had an accident
- Suffer from nausea or dizziness
- Experience the onset of a disease
- Notice any irregularities
- Have pains of unknown reasons
- Suffer from diabetes
- Suffer from convulsive disorders

- Have your period
  - Have metal in patient's body
10. Keep out of the reach of children.
  11. It is contraindicated for use on any muscle that is injured or diseased.
  12. Powered muscle stimulators should be used only with the leads and electrodes recommended for use by the manufacturer.
  13. electrode placement and stimulation setting should be based on the guidance of the prescribing practitioner.
  14. some patients may experience skin irritation or hypersensitivity due to the electrical stimulation or the conductive medium. The irritation can usually be reduced by using an alternative conductive medium or alternative electrode placement.
  15. Please consult a professional to replace the battery.
  16. No modification of this stimulator is allowed.
  17. That stimulation should not be applied directly on the eyes, covering the mouth, (especially the carotid sinus), or from electrodes placed on the chest and the upper back.
  18. WARNING:
    - Please don't use the accessories, detachable parts, and materials not described in the instructions for use
    - No modification of the equipment

- Please don't use the equipment outside its carrying case when some part of the protection required by this standard is provided by that carrying case

19. Do not use this product with an electrocardiograph meter (ECG) or any other medical apparatus.

20. Do not use this product with any topically applied creams or ointments.

21. The device should not be used while the user is connected to high-frequency surgical equipment, it may cause burn injuries on the skin under the gel electrodes, as well as problems with the device.

22. Do not use in the vicinity of shortwave or microwave therapy equipment, since this may affect the output of the device.

23. Do not use on a wet body.

24. Do not use in the bathroom or in any other area of high humidity as this may cause an uncomfortable intense stimulation.

25. Do not use while driving or operating any other mechanical equipment.

26. If the electrode was not intact, do not use the device. Degraded sensor and electrodes, or loosened electrodes may cause output performance with potential risk.


27. Additional precaution:

- Keep the device out of the reach of children

- Potential allergic reactions to accessible materials used in the equipment, Electrode gel may cause a skin irritation, if this happens, please seek medical advice.

- Do not use on around area of the body with skin disease, damage or inflammation.

## 2. Operation of the device and Feature

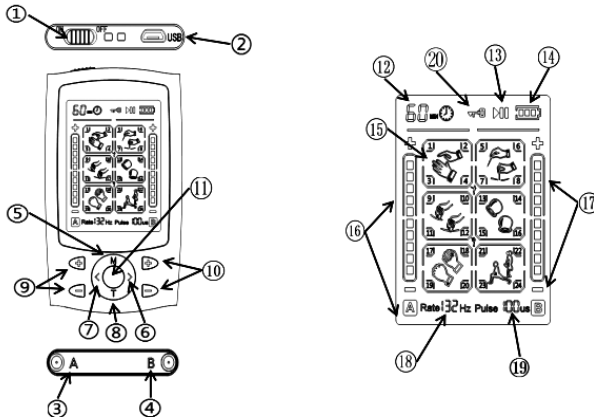1. On/ off button	2. USB port
3. A channel	4. B channel
5. Sub-mode selector	6. Main mode selector ("> ")
7. Main mode selector (" < ")	8. Time adjustment
9. A channel "+" / "-" intensity	10. B channel "+" / "-" intensity

11. Pause key/ Unlock	12. Time icon
13. Pause icon	14. Battery icon
15. Modes icon	16. A channel intensity icon
17. B channel intensity icon	18. Frequency icon
19. Pulse width	20. Locked

## Operating Procedures

**Note:** Unit may need to be charged for up to 10 hours after battery runs out for the first time. The charging process will last approximately 1 hour after that.

1. Connect the wire to the port at the bottom of the unit.
2. Connect the pads to the wire by snapping them on.
3. Remove the protective film and place pads where the massage sensations are desired.  
Note: adding a few drops of water to the pads will increase their longevity.
4. For additional relief use the second port and repeat procedure to put on another two pads.

**Make sure both pads on the skin and not overlapping. Make sure your skin is clean.**

## 5. Turn on the device

Turn the device on by sliding the on/ off switch on the top of the unit. The unit will be activated.

## 6. Choose mode

Remember the strength will default to the 0 when changing the mode.

Press the ">" button to choose the main mode from by Clockwise, press the "<" button to choose the main mode counterclockwise.

Press the "M" button to choose the sub-mode in circle.

## 7. Set the intensity

Once you select a mode, gradually increase the intensity by pressing the "+" button; decrease it by pressing "-" button.

When you use the lead wire with 2 buttons, please make sure 2 pads are stuck to your skin, then you can adjust the intensity.

When you use the lead wire with 4 buttons(2 white and 2 blue buttons), please make sure at tach at least one blue on your skin.

Otherwise, you can not adjust the intensity. Blue wire is paired with white wire.

## 8. Switch the channel A/B

The default channel is channel A, press the "+/-" on the right to wake up the channel B, then you can repeat step 6 and 7.

## 9. Set the treatment time

Increase 10 minutes by pressing “T” button. Available application range is 10-60 minutes. The time remaining is automatically counted down. Then the unit will automatically turn off after the time runs out.

## 10. Pause the treatment

Please press the middle round button to pause the treatment when changing the massage point during the treatment, press it again to cancel the pause.

## 11. Lock the device

The device will be locked after 180 seconds of inactivity, the  icon will appear on the screen in locked state. You can Long press the middle round button for 2 seconds to unlock the device

**Note:** The back light will be off automatically after 30 seconds of inactivity.

12. Sit down and enjoy the deep smoothing sensation.

**Note:**

- Never connects this product with common headphone.
- Please do not touch the USB port in the process of using it.

The USB port is only used for connect the charger, do not

connect other device.

- The charger supplied by the manufacturer must in compliance with IEC/EN 60601-1, the use of unauthorized charger can degrade minimum safety.

### **13. Turn off the device**

Push the switch button on the bottom from ON to OFF to turn off the device. The device will automatically turns off when the time runs out.

**14.** Remove the pads from you skin gently, and paste the plastic dust-proof film on them, take them off the electrode wires and unplug the wires, put them and the device into the storage bag.

**Note:** Please don't tear off the pads vigorously.


Please don't remove the pads before turning off the device.

### **Features**

- 2 independent output channels with 24 different modes
- 20 intensity levels
- 20 minutes default timer, can be adjustable from 10-60 minutes
- Internal rechargeable lithium battery
- Battery indicator to know when need to recharge battery
- large, easy to read LCD display
- Key lock function protection
- Memory function

- ABS +stainless
- 2 years guarantee
- AC adapter 100-240V, USB Charger 5V
- Stimulator pads and pad wires are applied part

### Battery Charger Information

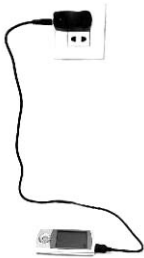1	General technological characteristics	- Input	100-240V AC 50/60Hz 0.25A
		- Output	DC5V --- 550mA
2	Complied standards	EN 60950 ;	
3	How it is connected to the device		

### Battery Information

Capacity: 180mAh

Voltage: 3.7V

Restriction: 4.2V

### 3. Product Specification

Name of product	Tens and Muscle Simulator
Model No.	AS8012B
Product material	Shell material of controller with ABS+stainless Pads with silica gel
Power/voltage	3.7V
Consumed current	40mA
Status adjustment	24 kinds of status
Strength level adjustment	20 sections
Timer	With 10-60 minutes stop timer
Weight of unit	0.062 Kg
Normal working ambient temperature	5~40°C
Normal working ambient humidity and Atmosphere pressure	≤80%RH; Atmosphere Pressure: 700 ~1060hPa
Store and transport ambient temperature	-20~55°C
Store and transport ambient humidity and Atmosphere pressure	≤93%RH; Atmosphere Pressure: 700 ~1060hPa

Safety classifications	internal powered equipment, Type BF applied part, not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide, continuous operation
Charger	Input:100-240V 50/60Hz 0.25A Max Output: DC5V ---550mA Class II, not applied part, not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide, continuous operation
Software version	Version: 2014

## 4. Modes Description

There are 24 modes to choose from. The “M” button cycles through the 6 main modes, all of these 6 main modes contain different built-in 4 modes.

Main Mode	Mode No.	Description
Tuina	1 EMS	Relieves the muscle tension, promote blood flow back to the joint and improves

		your mobility.
	2 TENS	Stimulates circulation to end stasis, warms the channels to relieve pain and eliminate toxins from the body. Scraping the skin can stimulate one's immune system and then help to remove the disorder.
	3 EMS	Relief of pain & muscle tension. Relieves the tension, brings blood flow back to the joint and improves your mobility.
	4 EMS	Helps reduce muscle spasms, increase the range of motion, injury recovery and the after surgery recovery.
Acupunc-t ure	5 TENS	It helps chronic diseases such as insomnia, constipation, headaches, and the tension associated with stress.
	6 TENS	This mode is used for the points on the Chinese meridian pressure point chart to assist headaches, a variety of ailments as well as muscle and joint pain. Particularly good for fast relief of acute pain.
	7 EMS	Provides the user with scratching and kneading sensation. Releases unhealthy elements from injured areas and


		stimulates blood flow and healing.
	8 TENS	This mode is used for the points on the Chinese meridian pressure point chart to assist headaches, a variety of ailments as well as muscle and joint pain. Particularly good for fast relief of acute pain.
Guasha	9 TENS	Releases unhealthy elements from injured areas and stimulates blood flow and healing
	10 TENS	It has a soothing and relaxing effect, improves blood and lymphatic flow, and corrects minor misalignments of muscles, tendons and ligaments
	11 TENS	Provides the user with a kneading and beating sensation. Releases unhealthy elements from injured areas and stimulates blood flow and healing.
	12 TENS	Provides the user with a beating and scraping sensation.
Cupping	13 EMS	It is the manipulation of superficial and deeper layers of muscle and connective tissue using various hands-on techniques, to enhance physical function, aid in the healing process, release tension, and

		promote well-being.
	14 TENS	It's a form of therapy in traditional Chinese medicine. Cups are placed on the particular section of the body to create suction often using heat. The method draws out toxins, mobilizes blood flow, alleviates muscle pain and even improves the quality of sleep.
	15 TENS + EMS	This form of therapy involves hands-on techniques to increase circulation, relieve tension, reduce stress, relieve anxiety, improve the quality of sleep, and promote relaxation throughout the entire body, as well as many other benefits
	16 EMS	The appliance puts out hard thumping sensation that massage your muscles, which can give you an invigorated and revitalized feeling.
Tapping	17 TENS	For muscle strains, herniated discs, stenosis, spondylosis, and general arthritis pain.
	18	For relieving back and neck pain, stiff

	TENS	muscles, anxiety, fatigue, migraines, rheumatism, and even cellulite etc.
	19 TENS	It is a type of alternative medicine consisting of finger and palm pressure, stretches, and other massages techniques.
	20 EMS	Enhance function, aid in the healing process, inhibit motor-neuron excitability, decrease muscle reflex activity, promote relaxation, recreational and well-being
Combina-t ion	21 EMS	This mode is effective for relieving joint pain, sciatica, muscle spasms, and pain in the back, neck, and shoulders.
	22 EMS	Enhancing body fluid circulation, relieving muscle tension and pain, promoting metabolism in the body and cleansing the body.
	23 TENS	Helps reduce muscle spasms, increase the range of motion, injury recovery and the after surgery recovery.
	24 TENS	The appliance puts out quick and strong pulses in a regular rhythm; good for all major muscle groups.


## 5. Placement of Electrodes

Neck Pain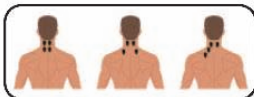


Suggested Mode:  
Mode 2 5 6

Chronic Hip Pain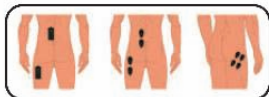


Suggested Mode:  
Mode 1 3 4

Frozen Shoulder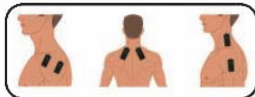


Suggested Mode:  
Mode 2 6 8 9

Lower Back Pain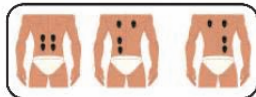

Suggested Mode:  
Mode 7 13 16

Tennis Elbow

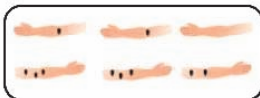
Suggested Mode:  
Mode 9 10 11 12 14

Carpel Tunner Syndrome

Suggested Mode:  
Mode 17 18 19 23 24

Stomach Pain

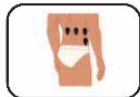
Suggested Mode:  
Mode 20 21 22

Erector Spinalis

Suggested Mode:  
Mode 1 4 7

### Sciatica

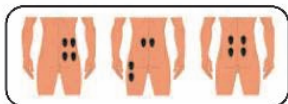Suggested Mode:  
Mode 2 5 6 8 9 10

### Knee Pain-Post OP

Suggested Mode:  
Mode 1 4 7 13 15

### Dedenerative Arthritis-Knee Pain

Suggested Mode:  
Mode 15 16 20 21 22

### Lower Leg Pain

Suggested Mode:  
Mode 16 20 21 22

### Foot Pain

Suggested Mode:  
Mode 1 3 7 13

### General System Nerves, Muscle, Tierdness&Bad Blodd Circulation

Suggested Mode:  
Mode 15 16 20 21 22

### Ankle Pain

Suggested Mode:  
Mode 1 3 4 7

## 6. Troubleshooting

Problem	Cause	Solution
One pad feels stronger than the other.	This is normal. Different areas of your body will react differently.	Nothing needs to be done. Make sure the pads are moist and are making good contact.
During the massage, the skin feels a painful burning sensation or the stimulation becomes weakened.	Adhesive gel pads are not adhering firmly to the skin. Or the gel pads are too dry.	Apply a few drops of water to adhesive surface of each pad and make sure the pads are pressed firmly to the skin during application.
Unit is on, but no sensation is felt in the pads.	Both pads are not firmly on the skin. Or there is a loose connection. Or the intensity level needs to be increased.	Make sure both pads are firmly pressed to the skin. Make sure all connections are secure from the unit to the wires and the wires to the pads. Increase the intensity level.

<p>Adhesive gel pads do not stick to skin even after cleaning and moistening the gel pad.</p>	<p>Adhesive gel pads need to be replaced.</p>	<p>Replace the set of pads. Pads can be purchased from local distributor.</p>
<p>Unit does not turn on or the screen is dim.</p>	<p>Battery is low.</p>	<p>Recharge the battery.</p>
<p>Does not provide adequate pain relief.</p>	<p>Not using it long enough. Not lying down then massaging neck.</p>	<p>Use your Stimulator 20-30 minutes at a time, 3-6 times per day. Lie on your back for neck support when you need relief for headaches or neck pain.</p>
<p>The hand is electrocuted when touching the pads</p>	<p>The intensity is higher than 0 while the device isn't paused</p>	<p>Pause the device when changing the massage point or increase the intensity to 0.</p>

## 7. Maintenance and Storage

1. Unplug the output cord from the output jack of the controller after using.
2. Cover both pads with the protective film before storage.
3. Never fold the massage pads.
4. To keep the controller clean, use a soft and dry cloth for dust or a soft damp cloth for any dirt and smudges. Do not use any cleaning solutions to clean the controller and its pads.
5. Never apply the pads on any surface other than your skin. If the pads become soiled or dirty, the adhesive power may decrease. In this case, moisten the surface of the pads with water and wipe away the dirty portion. This will allow a temporary restoration of the adhesive power. However, too much water will result in loss of the adhesive power.
6. Do not use or store the devices where there are magnetic fields or electric wave (near TV set or speakers).
7. Do not place the devices in areas of high temperature, high humidity, or under direct sunlight.
8. Keep the device out of reach of children.
9. All worn accessories should be dealt according to the regulations.
10. Do not immerse the device in water or any liquid. Do not drop the device or throw it from a height.
11. After using the device, please remove the electrode and replace the protective film. Always use the protective film when the

electrode is not in use.

## 12. **Cleaning**

- Machine washing and soaking is PROHIBITED.
- Do not wash the belt in fabric softener as this can damage the belt.
- Do not dry clean.
- Do not use bleach.
- Clean accessories with a slightly wet sponge with detergent and wipe with a cloth.
- Clean the unit with a cloth soaked in with disinfectant (Mercryl type) diluted in water at 1/10th.
- Clean the electrodes with a mix of water/hypoallergenic detergent, then rinse thoroughly.
- Keep the device and electrode clean

## 8. **Electromagnetic Compatibility**

The device has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices to IEC 60601-1-2: 2014. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

### CAUTION:

Do not use this device simultaneously with devices having high EMI

levels.

<b>Guidance and manufacture's declaration – electromagnetic emission</b>		
This device <i>is</i> intended for use in the electromagnetic environment specified below. The user of AS8012B should assure that it is used in such an environment.		
<b>Emission test</b>	<b>Compliance</b>	<b>Electromagnetic environment – guidance</b>
RF emissions CISPR 11	Group 1	This device <i>use</i> RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	This device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that
Harmonic emissions IEC 61000-3-2	Class A	

Voltage fluctuations/ flicker emissions  IEC 61000-3-3	Complies	supplies buildings used for domestic purposes.
--	----------	--

Guidance and manufacture's declaration – electromagnetic immunity			
This device is intended for use in the electromagnetic environment specified below. The customer or the user of AS8012B should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD)  IEC	±8 kV contact  ±15kV air	±8kV contact  ±15 kV air	Floors should be wood, concrete or ceramic tile. If floor is covered

61000-4-2			with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	$\pm 2$ kV for power supply lines	$\pm 2$ kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	$\pm 1$ kV line(s) to line(s)	$\pm 1$ kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruption	<5% UT (>95% dip in UT)	<5% UT (>95% dip in UT)	Mains power quality should be that of a

<p>ns and voltage variations on power supply input lines</p> <p>IEC 61000-4-1 1</p>	<p>for 0.5 cycle</p> <p>40% UT</p> <p>(60% dip in UT)</p> <p>for 5 cycles</p> <p>70% UT</p> <p>(30% dip in UT)</p> <p>for 25 cycles</p> <p>&lt;5% UT</p> <p>(&gt;95% dip in UT)</p> <p>for 5 sec</p>	<p>for 0.5 cycle</p> <p>40% UT</p> <p>(60% dip in UT)</p> <p>for 5 cycles</p> <p>70% UT</p> <p>(30% dip in UT)</p> <p>for 25 cycles</p> <p>&lt;5% UT</p> <p>(&gt;95% dip in UT)</p> <p>for 5 sec</p>	<p>typical commercial or hospital environment. If the user of this device requires continued operation during power mains interruptions, it is recommended that this device <i>be</i> powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50Hz/60H</p>	<p>30A/m</p>	<p>30A/m</p>	<p>Power frequency magnetic fields</p>

z) magnetic field IEC 61000-4-8			should be at levels characteristic of a typical location in a typical commercial or hospital environment.
--	--	--	---

NOTE: UT is the a.c. mains voltage prior to application of the test level.

 <p><b>Disposal of Materials</b></p>	Do not dispose this product as you would other household waste. Dispose this device in accordance with the corresponding local regulations. - Electrical and electronic devices contain hazardous substances that can have harmful effects on the environment and/or human health and should be recycled properly.
---	--

## 9. Warranty

1) The warranty period for device is two year from date of delivery. In case of a warranty claim, the date of delivery has to be proven by means of the sales receipt or invoice.

2) Repairs under warranty do not extend the warranty period.

3) The following cases are not included in the warranty.

- All damage which has arisen due to improper treatment, e.g. nonobservance of the user instruction.

- All damage which is due to repairs or tampering by the customer or unauthorized third parties.

- Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.

- Accessories which are subject to normal wear and tear.

4) Liability for direct or indirect consequential losses caused by the unit is excluded even if the damage to the unit is accepted as a warranty claim.

5) Anything about the device, please feel free to contact [support@belifu.com](mailto:support@belifu.com).

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