



## Ultra-Wide Microplush Heating Pad



### OPERATION MANUAL

READING BEFORE USING

**Model: NA-H1821B**

## **ACTIVATE YOUR 24 MONTH WARRANTY & EXTEND WARRANTY TO 36 MONTH**

Register within 2 weeks after receiving your heating pad, extend your warranty to 5 years.

Make sure you do this NOW to ensure your purchase is 100% covered.

**PLEASE VISIT NOW**

→ <https://belifu.com/warranty>

### **CUSTOMER SERVICE**



support@belifu.com



@Belifu



WhatsApp +86 19986945097

Please download the user manual on below link:

<https://belifu.com/downloads>

# IMPORTANT SAFETY INSTRUCTIONS DO NOT DESTROY

When using this or any other electrical appliance, always follow these basic safety precautions:



**DANGER: TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK AND FIRE, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS.**

1. READ ALL INSTRUCTIONS.
2. DO NOT USE WHILE SLEEPING.
3. NEVER LEAVE THIS APPLIANCE UNATTENDED, ESPECIALLY IF CHILDREN ARE PRESENT.
4. DO NOT USE ON AN INFANT.
5. DO NOT USE ON AN ANIMAL.
6. THIS PAD IS NOT SUITABLE FOR THOSE PERSONS, WHO ARE WITH POOR BLOOD CIRCULATION OR UNCONSCIOUS PERSON.
7. THIS PAD IS NOT TO BE USED ON A PERSON WITH DIABETES.
8. DO NOT USE ON AREAS OF INSENSITIVE SKIN.
9. BURNS MAY OCCUR REGARDLESS OF THE CONTROL SETTINGS, PLEASE CHECK THE SKIN UNDER PAD FREQUENTLY.


10. DO NOT USE PINS OR OTHER METALLIC THINGS TO FIX THE PAD.
11. DO NOT SIT ON OR PRESS THE PAD.
12. PUT THE PAD ON TOP OF THE BODY, NOT UNDERNEATH THE BODY. BECEASE THE HEAT LEVEL MAY INCREASE, WHEN HEAT IS TRAPPED.
13. NEVER USE LINIMENTS, OINTMENTS OR SALVES TO THE SKIN, ESPECIALLY THOSE CONTAINING HEAT-PRODUCING INGREDIENTS, WHEN USING PAD, AS SKIN BURNS COULD RESULT.
14. NEVER PULL THIS PAD THROUGH THE SUPPLY CORD.
15. NEVER UNPLUG THE PAD FROM THE ELECTRIC SOCKET WITH A WET HAND.
16. IF USING AN EXTENSION CORD FOR THIS PAD, THE MARKED ELECTRICAL RATING OF THE EXTENSION CORD SHOULD BE AS GREAT AS ELECTRICAL RATING OF THE PAD (SEE ELECTRICAL SPECIFICATIONS SECTION). THIS PAD CAN BE USED ONLY ON A 120 VOLT AC ELECTRICAL OUTLET. UNPLUG THE PAD FROM SOCKET WHEN NOT IN USE.
17. DO NOT USE THE ELECTRIC CORD AS A HANDLE.
18. PLEASE EXAMINE THE INNER VINYL PAD CAREFULLY EVER TIME, BEFORE USING. PLEASE DISCARD THE PRODUCT IF THE INNER VINYL PAD SHOWS ANY SIGN OF DETERIORATION.
19. DO NOT BEND OR PINCH THE CORD, AND DO NOT USE THIS PAD IF IT HAS A DAMAGED POWER CORD OR PLUG, HAS MALFUNCTIONS, HAS BEEN DROPPED, OR IS DAMAGED. THIS PAD SHOULD NOT BE REPAIRED BY YOURSELF. IF FOR ANY REASON THIS PAD DOES NOT FUNCTION SATISFACTORILY, PLEASE FOLLOW THE WARRANTY IN SERVICE INFORMATION.
20. DO NOT STEP ON THIS PAD IN ANY WAY.
21. COMBINE THE CORDS INTO A BUNDLE LOOSELY, WHEN STORING, TIGHT WRAPPING MAY DAMAGE THE CORD AND THE INTERNAL PARTS.
22. THIS PRODUCT IS INTENDED FOR PERSONAL USE, BUT NON-COMMERCIAL, NON-INDUSTRIAL USE. DO NOT USE THIS PRODUCT OUTDOORS OR FOR ANY OTHER PURPOSE.
23. THIS HEATING PAD HAS A POLARIZED PLUG (ONE BLADE IS WIDER THAN THE OTHER), CO NSIDERING ABOUT SAFETY. THIS PLUG WILL FIT INTO A POLARIZED OUTLET ONLY ONE W AY. IF THE PLUG DOES NOT FIT FULLY INTO THE OUTLET, REVERSE THE PLUG. IF IT STILL D OES NOT FIT, PLEASE CONTACT WITH A QUALIFIED ELECTRICIAN.  
DO NOT ATTEMPT TO MODIFY THE PLUG OR DISABLE THIS SAFETY FEATURE IN ANY WAY.
24. SAVE THESE INSTRUCTIONS.

## **INTRODUCTION**

Thank you for purchasing Belifu Heating Pad. We designed this wide size heating pad to provide soothing heat therapy to large muscle groups in your back, legs, arms and more.

## USING BELIFU HEATING PAD

Attach the pad Connector to the Control Connector. Push both parts together securely to ensure a tight fit.

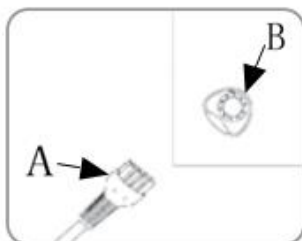

A: Control Connector

B: Pad Connector

NOTE: The Control Connector must be connected to the Pad Connector before plugging the unit into an electrical outlet.

1. After connecting the Pad Connector to the Control Connector, plug the unit into a 120V electrical outlet.

2. To turn the heating pad on by press the power button. Continue to press the power button to select the desired heat setting:

L: Low

M: Medium

H: High

3. To turn heating pad off, press the power button one more time.

NOTE: If the unit does not turn on when you press the power button, make sure the Control Connector is securely plugged into the Pad Connector.

## AUTO-OFF FEATURE

To conserve energy and prevent skin burns caused by excessive use. Belifu introduces an auto-off feature. The unit will turn off automatically after 90 mins of continuous use. After a power outage, Belifu automatically resets the timer on the auto -off feature.

## WARNING

Do not use this product if the Controller or Pad Connectors are wet. Make sure the product connections are completely dry before using.

1. Place the moistened side of the Pad on top of the body.

2. Turn the product off by pressing the On/Off Button and unplug it when it is not in use.

## WASHING INSTRUCTION

**DO NOT DRY CLEAN! DO NOT USE BLENCH! DO NOT USE WRINGER!**

1. Disconnect the power cord from the electrical outlet.
2. Disconnect the heating Pad with the Control Connector, through pulling the Control Connector from the Pad connector.
3. The Pad without Controller can be washed by washing machine or hand, using a mild detergent, cool water, and a gentle cycle.
4. Dry with a rope or lay the pad flat to dry, or tumbled dry on low heat.
5. Make sure the product is completely dry before the next use, especially the connector of the pad should be totally dry.
6. To store the product, please loosely roll the Heating Pad together, loop the cord over the rolled ends, and then store it in a cool, dry, dark location after each use. Avoid sharply folding the Heating Pad.

## ELECTRICAL SPECIFICATIONS

Voltage	Frequency	Wattage	Amperage
120 VAC	60 Hz	85W	0.7A

## FREQUENTLY ASKED QUESTIONS

### 1. Does it have even heat on both sides? Or one side more than the other?

Both sides of the heating pad heat evenly. There is a plush side and a standard cloth side. One side is plush and soft and more comfortable for direct skin contact, but either side will work. The non-plush side has written safety and washing instructions. You could use the standard side against a shirt or towel. It does transfer a bit more heat than the plush side.

### 2. Does the cover remove for washing?

This heating pad does not have a removable cover. The Washing instructions are included in the booklet you get, please read carefully before washing. It is better to sew one or modify a pillow case if you want the feel of cotton and reduce how often you would wash the heating pad.

### 3. What temperature does this product reach?

Maximum temperature does not exceed the Underwriters Laboratories temperature limit of 176 °F (80 °C), when set to “Hi”. Temperature varies by heat setting; you can use a lower heat setting, but depending on the specific application and your personal preference.

### 4. How much time does it take to warm up to each temperature?

Temperature Selection	Indicator	Heat Up Elapsed Time
Low	1	6 min
Med	2	8 min
High	3	10 min

## WARRANTY

Belifu comes with an industry-leading 2 Year Warranty that begins on the date of purchase.

The warranty applies to the heating pad and necessary parts and labor relating thereto. The warranty does not apply to the damage resulting from failure to following the operating instructions, accidents, abuse, alterations, or disassembly by unauthorized individuals.

## PRODUCT REGISTRATION

To ensure your full warranty and to receive product updates and streamlined customer support, remember to register your product at:

→ <https://belifu.com/warranty>

## PRODUCT SUPPORT

If you have product questions or concerns, don't hesitate to contact us directly at:



[support@belifu.com](mailto:support@belifu.com)



@Belifu



WhatsApp +86 19986945097

Belifu has an award-winning warranty, exchange, and customer service program that guarantees hassle-free solutions to any issue you might have within 24 hours!

## FCC NOTICE

Note: This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

<https://belifu.com/warranty>

[support@belifu.com](mailto:support@belifu.com)

此页无需打印

材质：105g 铜版纸，A5 纸张大小，黑白印刷